**CHAMPIONS®
HYA-LIQUID**

D

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GB

INSTRUCTIONS FOR USE

CHAMPIONS® HYA-LIQUID is a biologically inert product on the basis of hyaluronic acid of non-animal origin.

On account of its structure and processing and the low molecular weight of hyaluronic acid, it shows a quick absorption in the area of the oral mucosa. This is a desired effect for the treatment of changes in the oral cavity such as e.g. stomatitis, gingivitis, ulcers and aphthae.

CHAMPIONS® HYA-LIQUID is a clear, sterile solution made of hyaluronic acid. Hyaluronic acid has been clinically proven for the treatment of various changes in the oral mucosa and the periodontium.

COMPOSITION

1 ml **CHAMPIONS® HYA-LIQUID** contains

Sodium hyaluronate	14 mg
Sodium chloride	6.9 mg
Water for injection	1 ml

THERAPEUTIC INDICATIONS

- regeneration of the oral mucosa and the periodontium

- for the supportive treatment of peri-odontosis
- gingivitis and stomatitis
- aphtae and ulcers
- after dental extractions

INSTRUCTIONS FOR USE

CHAMPIONS® HYA-LIQUID is applied into the area to be treated using the special needle (ANEL curved 27G) included in the package. The application has to be repeated depending on the treatment success.

CONTRAINDICATIONS

Do not use

CHAMPIONS® HYA-LIQUID in case of sensitivity to hyaluronic acid. According to clinical literature, however, this occurs in > 1/2000 cases.

PRODUCT SAFETY

On account of its composition –

CHAMPIONS® HYA-LIQUID acid of non-animal origin – provides a safe use for the indications listed above.

It is a sterile medicinal product without addition of auxiliary agents such as e.g. preservative agents.

No interaction with drugs (e.g. antibiotics) is known.

MANUFACTURER

BioScience GmbH
Rheinstr. 96
D-56235 Ransbach-Baumbach, Germany
www.bio-science.org

MARKETING

Champions-Implants GmbH
Bornheimer Landstr. 8
55237 Flonheim/Germany
info@champions-implants.com
www.champions-implants.com

SYMBOLS

	Moist heat sterilization
	Do not use if the packaging is damaged
	Needle sterilization with EO
	Store in a dry place
	Do not reuse
	Do not expose it to direct sunlight
	Refer to instruction for use
	Order number
	Storage temperature
	Expiry date
	Manufacturer
	Charge

CE 0297 CE label in accordance with MDD 93/42/EEC 0297 is the number of the „notified body“

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